# Futura & Nova

Dansk	Fiberdynen Nova & Futura brugsanvisning <b>4-5</b>
Norsk	Fiberdynen Nova & Futura brugerhåndbok <b>6-7</b>
Suomi	Kuitu peitto Nova & Futura Käyttöohjeet <b>8-9</b>
English	Fiber quilt Nova & Futura Owner's Manual <b>10-11</b>
Deutsch	Fasersteppdecke Nova & Futura Bedienungsanleitung <b>12-13</b>
Español	Edredón de fibra Nova & Futura Manual del propietario <b>14-15</b>
Français	Manuel du propriétaire duvet de fibres Nova & Futura <b>16-17</b>
Italiane	Fibra piumone Nova & Futura Manuale dell'utente <b>18-19</b>
Nederlands	Fiber dekbed Nova & Futura Nederlandstalige handleiding <b>20-21</b>
Polski	Kołdra włókno Nova & Futura Podręcznik użytkownika <b>22-23</b>
Česká	Vlákno deka Nova & Futura Uživatelská příručka <b>24-25</b>
日本語:	ファイバーキルト Nova&Futura 取扱説明書 <b>26-27</b>

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## – softness and security

### A calming cognitive aid

The Fibre Quilt is a weighted quilt that stimulates the senses and acts as a cognitive aid. It is unique. It is soft and naturally enveloping, and provides an even pressure around the body. Users find that the quilt has a calming and relaxing effect.

Occupational therapists and physiotherapists, carers, physicians and people who have used the Fibre Quilt in practise confirm that it is soft and feels more or less like a regular quilt. When you use the Fibre Quilt, you do not feel that it is an aid. It is also great for sitting or lying on.

### Who needs a weighted quilt?

Clients who find it difficult to rest will often find the Fibre Quilt is a real godsend – night and day.

The Fibre Quilt may offer relief to clients with cognitive disorders who have an impaired sense of time and space.

Anyone who needs a comfortable, slightly weighted blanket is well-advised to choose the Fibre Quilt. It is soft and comfortable and fosters a sense of calm.

## PRACTICAL INFORMATION

### Washing instructions, Fibre Quilt

The Fibre Quilt is hygienic and machine washable at 60° C. Tumble-dry at low temperature. Also suitable for dry cleaning.

### Hygiene cover Ecoverde®

The completely waterproof hygiene cover made of cotton and polyurethane helps keeping the quilt clean and dry. The barrier layer ensures a comfortable bed environment because it releases body heat and moisture.

Sizes available: 140 x 200 cm/150 x 200 cm (large) and 110 x 125 cm (Futura)/110 x 140 cm (Nova) (small). Wash: Max 95°. Tumble dry: Low heat. Dry Cleaning: No. Ironing: Yes. Flame retardant: Yes

### Use

One side of the Fibre Quilt is quilted and softer than the other. Use this side close to the body.

### Protect your quilt

To protect your quilt and prolong its lifetime, we recommend that you use a quilt cover. A specially designed hygienic quilt cover is available as an accessory. It is flame-resistant and water-repellent.

### Separate the quilt into two halves before washing/tumble-drying

The Fibre Quilt has a zip fastener down the middle. You can therefore wash each half separately in an ordinary household washing machine. NB: Do not exceed the machine's maximum load capacity.

### Transportation

The Fibre Quilt is supplied in a very practical travel bag.

### Dimensions and weight

#### Fibre Quilt, large

140 x 200 cm/150 x 200 cm, weight from 4 to 14 kg

#### Fibre Quilt, small

110 x 125 cm (Futura)/100 x 140 cm (Nova), weight from 2.3 to 5 kg

### Choosing the right weight

It is often difficult to gauge precisely how much weight the individual client needs. Consider the effect you seek to achieve and how the Fibre Quilt will be used.

### Technical data

#### Materials

The Fibre Quilt is made of Trevira and polyester fibre

#### Safety

The Fibre Quilt and hygienic quilt cover are flame-resistant. (EN-ISO 12952-1-2:2010 & BS 5852:2006)

#### Country of Origin

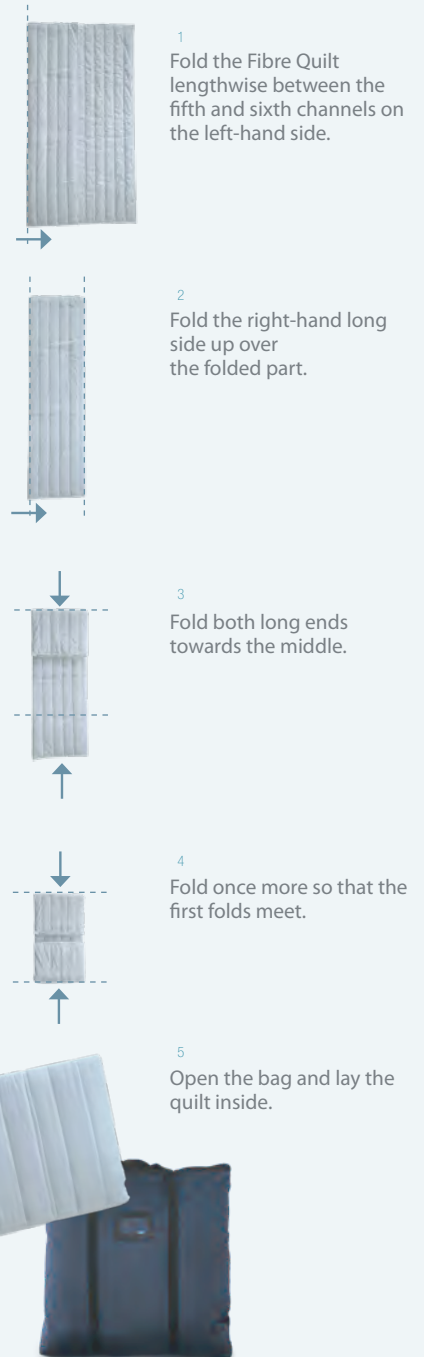
The Fibre Quilt is made in the EU. Novista AB, Johannebergsvägen 6 SE-312 72 Skummeslövsstrand, SE.

#### Labelling

The Fibre Quilt carries the CE-label. Medical Devices Directive 93/42/EU

The easy way to pack your Fibre Quilt in its travel bag:

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